



SPRAY PLASTER

INSTRUCTIONS FOR USE

SPRAY LIQUID PLASTER

70ML

Carefully read these instructions for use, as it contains important information for the patient about the use of the product Medicare Spray Plaster. Keep this leaflet in case there is a need to re-read.

WHAT IS MEDICARE SPRAY PLASTER?

Medicare Spray Plaster is an effective, transparent, air-permeable, elastic plaster intended for use on wounded skin.

It creates a mechanical barrier, which protects from the action of external factors.

WHAT IS MEDICARE SPRAY PLASTER USED FOR?

Wounded skin is vulnerable to actions of external factors like contaminations, too high humidity or micro-organisms, which can disturb the natural process of wound healing. The Medicare Spray Plaster is an excellent alternative for traditional sticking plasters, which are often tedious when used on mobile parts of the body.

Medicare Spray Plaster can be used in high humidity or dusty environments, where traditional sticking plasters don't fulfil their function. Medicare Spray Plaster creates an elastic film on the skin surface, allowing unrestricted movement. The transparent form of the plaster enables constant assessment of the wounds condition. The product is also waterproof, allowing it to stay on the skin surface even after washing and therefore does not require renewed application.

HOW TO USE

FOR EXTERNAL USE ONLY.

This medical device should be used according to the instructions for use.

Do not use on deep wounds with infraction of dermis.

Before using the Medicare Spray Plaster, thoroughly clean and dry the wounded skin. Shake the can and then spray the plaster onto the wounded skin from a distance of 10-15cm for 1-2 seconds. Leave until dry.

CONTRAINDICATIONS AND PRECAUTIONS

Do not use on mucosa of the eyes, nose and oral cavity.

Do not use on deep wounds with infraction of dermis.

Do not use in case of allergy (hypersensitivity) to product ingredients.

Do not use after the expiration date located on the packaging.

Do not use if the can is damaged.

DANGER

Extremely Flammable Aerosol.

Pressurised container: May burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn even after use. Protect from sunlight.

Do not expose to temperatures exceeding 50°C/122°F. Do not spray on an open flame or other ignition source. Keep out of the reach of children. Protect from getting product in the eyes. Do not inhale.

STORAGE CONDITIONS

Store in the original packaging in a dry, cold, covered, clean and dark place, out of the reach and sight of children.

Store at a temperature between 15-25°C.

COMPOSITION:

Dimethyl Ether, Alcohol Denat., Acrylates / Octylacrylamide Copolymer, Isopropyl Myristate.

PACKAGING DESCRIPTION

Medicare Spray Plaster is packed in an aluminium can with 70ml of medical device. Individual package includes: a box, a can and instructions for use.

Medicare Spray Plaster is a medical device.

MD500

Made in Poland

Rev.:21092021



GOFARM SP. Z O.O. SP. K.

Górników Street 21/26,
30 -819 Kraków, POLAND
WWW.GOFARM.PL

Manufactured for: FLEMING MEDICAL LTD

Limerick, V94 K6W4, IRELAND
1800 307777 (ROI) 0800 7314315 (UK)
WWW.FLEMINGMEDICAL.IE

SYMBOLS PRESENTED ON A PACKAGE	
	Manufacturer
	Expiration Date
	Batch Number
	Read Instructions for Use
	Storage Temperature
	Do not use if packaging is damaged
	The container of defined capacity
	Medical Device
	Recyclable Aluminium

Date of instructions for use update:
September 2021. MSP/IU/09/2021.