



MD1882
INFRARED EAR
THERMOMETER


How to Install a new Probe Cover.

- ⊗ Always use a clean, undamaged and new probe cover.
Be sure ear canal is clean. (Note: ⊗ “Do not reuse”)

(1) Attach the probe cover to the connection ring.

**** Make sure to place the connection ring over the smooth side of the probe cover as the following drawing.**

- (2) Align the center of probe cover with the center of probe.
(3) Push the connection ring until the click sound. This means the probe cover has been firmed tightly.

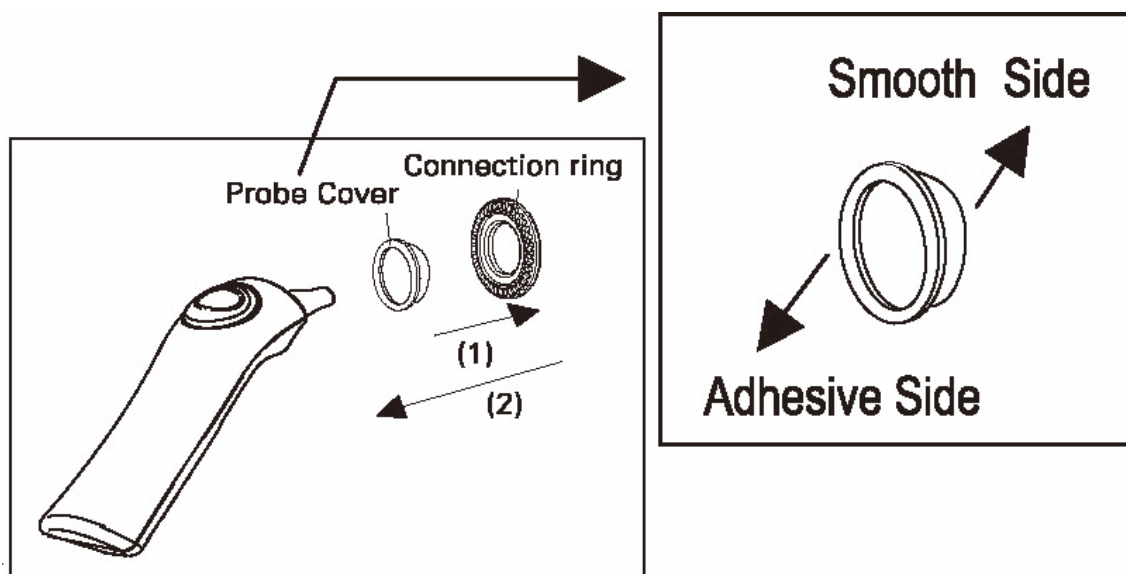

 **Warning: Keep the probe covers and connection ring away from children.**



Distributed by
Fleming Medical Ltd
Corcanree Business Park
Dock Rd, Limerick, Ireland
www.flemingmedical.ie



 **Radiant Innovation Inc.**
1F, No.3, Industrial East 9th Road,
Science-Based Industrial Park,
HsinChu, Taiwan 300

 **Medical Technology**
Promedt Consulting GmbH
Altenhofstrasse 80, D-66386,
St. Ingbert, Germany