



Fungal Nail Pen

Treat and Prevent



FACT BOX

Product Code:
MD1914

Product Category:
Footcare, OTC.

Product Approvals:
CE marked.

Composition:
Propylene Glycol, Aqua, Lactic Acid, Urea, Sodium Hydroxide.

Packaging: 4ml.

Treatment: Duration of treatment varies according to the severity of the condition and should be continued until a healthy nail has grown out. In most cases a new nail will grow in 6-12 months. One pen is sufficient for the treatment of one nail over 4 months.

Precautions: For external use only. Avoid contact with skin, eyes or mucous membranes as irritation may occur. In case of swallowing the product or in case of allergic reactions, consult a physician. Keep out of reach of children to prevent risk of choking on cap.

Medicare Fungal Nail Pen is a medical device that is used for the treatment of fungal nails. It improves the appearance of the nail and reduces discolouration. In most patients, the results are visible within 2 weeks. Optimal results are obtained when used daily until the healthy nail has grown out.

FEATURES

- Clinically proven application.
- Results after 1-2 weeks.
- Stops fungal growth.

DIRECTIONS:

Apply the solution once daily to clean and dry nail(s), preferably in the evening just before bed.

1. Remove the cap and rotate the end of the pen until the brush is damp (for the first use it can take up to 20 seconds).
2. Apply a thin layer on the affected nail(s), do not apply to the surrounding skin, this can cause skin irritation.
3. Replace the cap on the pen and allow the nail(s) to dry for a few minutes.

RECOMMENDED USE:

To prevent spreading of the infection, the pen should be used by one person and on the fungal nails only.

Barcode: 5099390416367