



Medicare Wart & Verruca Removal Pen

Clinically proven results
after 1-2 weeks



FACT BOX

Product Code:

MD1914

Product Category:

Footcare, OTC.

Composition:

Salicylic Acid
17.82%, Flexible Collodion
82.18% (Castor Oil, Dicaprylyl,
Hydroxypropylcellulose &
Ethanol).

Product Approvals:

Registered according to MDD
(Medical Device Directive,
93/42/EEC, Class IIa).

Shelf Life:

2 years from production
date, do not use after
3 months of first use.

Packaging:

2ml.

Precautions: Avoid contact
with eyes and exposure to
healthy skin. If irritation
occurs, discontinue use.

Medicare Wart & Verruca Removal Pen is a medical device for the treatment of warts and verrucas.

The active ingredient, salicylic acid, breaks down the wart causing it to gradually peel off. The solution also forms a mechanical barrier preventing the wart or verruca virus from spreading.

FEATURES

- Quick and easy application.
- Results after 1-2 weeks.
- No plasters needed.
- Contains salicylic acid to break down warts and verrucas.

DIRECTIONS:

Soak the affected area in warm water for five minutes and dry thoroughly. Twist the top of the pen slowly until the gel is released. Gently apply gel to the whole surface of the wart or verruca.

RECOMMENDED USE:

Use on warts and verrucas on hands and feet. Not suitable for children under the age of 4 years.

Barcode: 5099390416954



@Medicarefirstaid



@Medicarefirstaid



@Medicare1staid