

Timesco Topaz Deluxe Aneroid Sphygmomanometer

Product Code:
TIME202

CARE & MAINTENANCE:

- The cuff, tube and bulb are vulnerable to sharp instruments and chemical and corrosive liquid.
- Any unauthorised dismantling and reassembling on the manometer is forbidden.
- Do not drop the device or give it a violent mechanical shake. This may hinder the efficacy.
- Operate the device within the limit of authorised operation temperature.
- Keep the manometer away from longtime direct sunshine.



CLEANING:

- Wash the cuff with a moderate detergent and dry it.
- Wipe the tubes, bladder with a damp cloth.
- Wipe the manometer and chestpiece with dry and soft cloth.
- Do not use chemical agents and liquids to clean the unit as they may damage the device.

STERILISATION:

- Sterilisation is not necessary since the parts of the manometer should not come into direct contact with the patient's body during measurement. Wipe off the manometer and bulb with a damp cloth.
- Remove the bladder first and wipe the Velcro, bladder and tubes with a damp cloth. The cuff can be washed with soap and cold water like all the other cuffs, but you must rise the cuff with clear water afterwards and let them air dry.

STORAGE:

- Deflate the cuff completely before storage. Store the complete device in the bag provided. Temperature: -20°C to +70°C. Humidity: Relative humidity 85% (non-condensing).