



# Spacer for Inhaler

Maximise Your Medicine.  
Minimise Your Effort.



## FACT BOX

Product Code:  
MD1013

Product Category:  
Diagnostics - Respiratory Health

### Contents:

- 1 x Spacer Device
- 1 x Adult Mask
- 1 x Child Mask

### Caution:

For single-patient use only.  
Ensure mask fits securely over nose and mouth to avoid aerosol escape and reduced dosage.  
Store in a dry, dust-free environment away from prolonged sunlight.  
Inspect parts regularly; replace if damaged.

### Intended Use:

This device is intended for use by treating chronic obstructive pulmonary disease and asthma with medicine-assisted inhalation device. The device is a holding chamber, and is used in conjunction with medication sprays or metered dose inhalers (MDI) in the treatment of diseases of the airways.

The LifeSense® Breathe Spacer for Inhaler is a clinically designed holding chamber that ensures effective delivery of medication from a metered dose inhaler.

By holding the aerosol in the chamber for a moment, the spacer allows users to inhale medication more deeply and efficiently, improving respiratory outcomes for adults and children.

### FEATURES:

- Anti-static chamber for improved medication delivery
- Compatible with all standard Metered Dose Inhalers (MDIs)
- Adult and child masks included for versatile use
- Lightweight, portable design for everyday respiratory care

### DIRECTIONS:

1. Remove the cap from the metered dose inhaler (MDI), shake well, and insert it into the spacer's backpiece.
2. Ensure the chosen mask (adult or child) is securely fitted to the mouthpiece or use the mouthpiece directly.
3. Press the inhaler once to release the medication into the chamber.
4. Slow inhalation is recommended; if a whistling tone is heard, reduce breathing speed.

### CONTRAINDICATIONS AND PRECAUTIONS:

This device is a single-patient holding chamber designed for use with metered dose inhalers (MDIs). Do not share the spacer or masks between individuals. Check all components before use, do not use if cracked, damaged, or missing parts. Not made with natural rubber latex. Use only with inhalation medicines prescribed by a healthcare professional. Follow the medication's instructions for potential side effects of aerosol inhalation.

