



ICE SPRAY

Quick and easy application
to help relieve pain



FACT BOX

Product Code:
IMP01

Product Category:
Hot & Cold Therapy

Contents:
1 x 200ml Can

Specific safety requirements for the intended purpose:

May cause an allergic skin reaction in hypersensitive individuals. In case there is severe redness of the treated area, frostbite or blistering after excessive use of the spray and in case of doubt, always consult a doctor. Protect your eyes against the spray.

Do not inhale the spray. Extremely flammable aerosol. Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, an open flame or other sources of ignition. No smoking. Do not pierce or burn, even after use. Protect from sunlight.

Do not expose to temperatures exceeding 50°C. Keep out of reach of children. Any serious undesirable event associated with this product should be reported to the manufacturer, or to the state responsible agency for drug control or EUDAMED. Dispose of within organised waste collection.

Medicare Ice Spray is a cooling spray for quick pain relief associated with muscle contusion, joint sprain, muscle spasms and other injuries due to an impact fall or sprain and for pain relief during minor procedures such as venepuncture when inserting a cannula or catheter into the vein.

It contains low-boiling substances acting like a local anaesthetic on the affected site. It creates an immediate cooling effect on the treated site, relaxes the affected area and alleviates the feeling of pain for 3 minutes.

Medical device for quick pain relief.

INTENDED USER:

The entire population, children older than 3 years and diabetics.

DIRECTIONS:

1. Shake well before use.
2. Spray on the skin over the affected area at a distance of 15cm.
3. Repeat the application 2-3 times for about 5 seconds.

CONTRAINDICATIONS AND PRECAUTIONS:

Do not spray for prolonged time nor from a shorter distance to avoid frostbite. Do not apply to the face, neck, sensitive areas or open wounds. Do not use in case of an allergic reaction to any of the contained ingredients. Do not use in children under 3 years of age. Consult the use in pregnant and breastfeeding women with a doctor.

