



EAR SPRAY

Moisturises ear canal for clean, healthy ears



FACT BOX

Product Code:
MD507

Product Category:
Ear, Nose and Eye Care

Contents:
1 x 50ml Spray Can

Composition:
Aqua, Propylene Glycol, Polysorbate 80, Aloe Ferox Leaf Extract, Malva Sylvestris Extract, Phenoxyethanol, Ethylhexylglycerin, Tocopheryl Acetate, Parfum.

Warning:
Pressurised container: May burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C. Keep out of reach of children.



ALOE VERA & MALVA SYLVESTRIS

Cleanse, soothe, and protect your ears naturally with Medicare® Ear Spray, formulated with Aloe Vera and Malva Sylvestris. Designed for safe use by adults and children from age 3, this 360° spray softens earwax, hydrates the ear canal, and supports natural regeneration in a moist, gentle environment.

FEATURES:

- A 3-in-1 approach for daily comfort:
 - Softens and loosens earwax build-up.
 - Hydrates and soothes dry or irritated ear canals.
 - Supports regeneration.
- Suitable for all ages 3+ years and over and diabetics.
- 360° application for easy use.
- Naturally inspired ingredients: work in harmony with the ear's natural defences.

DIRECTIONS:

1. Tilt your head toward the opposite shoulder.
2. Gently insert the spray nozzle into the outer part of the ear.
3. Press the spray for 1 second, avoiding the eardrum.
4. Keep your head tilted for at least 30 seconds.
5. Wipe away any excess fluid with a clean tissue if needed.

CONTRAINDICATIONS AND PRECAUTIONS:

Do not use in case of allergic reaction to any of the ingredients of the product. If you have had paracentesis (puncture of the ear drum), use the spray only under medical recommendation. Consult your ENT specialist if you have: total blockage of the auditory canal by earwax, infection and pain in the ear or auditory canal, balance disorders, immunosuppression, surgery of the or auditory canal, ear drum rupture, foreign object in the auditory canal, pregnancy and lactation. Protect your eyes against the spray. Do not inhale the spray.

