



BURN SPRAY

Soothing hydrogel spray
for minor burns & scalds



FACT BOX

Product Code:
MD508

Product Category:
Burn Care

Contents:
50ml Spray Can

Composition:
Aqua, Sodium Hyaluronate, Propylene Glycol, Calendula Officinalis Flower Extract, Carbomer, Aloe Barbadensis Leaf Juice, Phenoxyethanol, Ethylhexylglycerin, Sodium Hydroxide.

Precautions:
In case of excessive redness of the skin and blisters formation after sunbathing, in the case of inflammatory response, or if in doubt, see a medical help. Also see a physician in case blisters keep expanding after the first treatment. Do not get in eyes. Avoid breathing the spray. Any serious adverse effect related to this product should be reported to the manufacturer and the Health Products Regulatory Authority. Pressurised container: May burst if heated. Keep away from heat, hot surfaces, sparks, open flames, and other ignition sources. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C. Keep out of reach of children. Dispose of the container within organised refuse collection. Use until the date indicated on the bottom of the container.

Treat minor burns and sun-exposed skin quickly and gently with Medicare® Burn Spray, a cooling hydrogel enriched with hyaluronic acid, aloe vera, and calendula.

Designed for home use, this spray forms a breathable protective film over the skin to soothe, hydrate, and speed up natural healing.

FEATURES:

- Cools and calms the skin instantly
- Supports fast healing with a moist, protective barrier
- Ideal for minor household burns and sunburn
- Safe for adults, children 3+, and diabetics

DIRECTIONS:

1. Cool the burn with cool running water for up to 15 minutes.
2. Apply spray on the whole burnt area.
3. Repeat the application twice a day at least to ensure that burnt area is constantly covered with protective film.
4. Rinse the actuator with running warm water after use. Place the lid back on so it does not dry up.

CONTRAINDICATIONS AND PRECAUTIONS:

Not intended for deep partial-thickness burns (Deep combustio bullosa) or full thickness burns (Combustio escharotica), when the epidermis and dermis is deeply damaged or full dermis is lost. Do not use in case of an allergic reaction to any of the contained ingredient. Do not use for children under 3 years of age. Consult a physician about the use in case of pregnancy or breastfeeding.