


Novel Coronavirus (COVID-19) Antigen Test Kit

Rapid results & easy to use

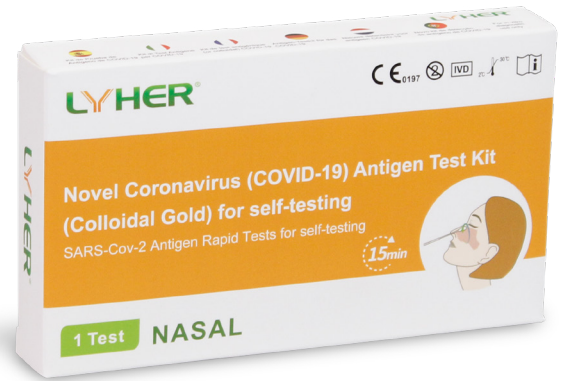

FACT BOX




Product Code:
COV01

Product Category:
Infection Control.

Product Approvals:
CE Approved.

Pack Size:
1 test per box.

Results:

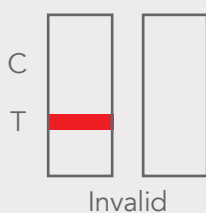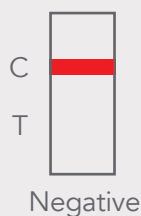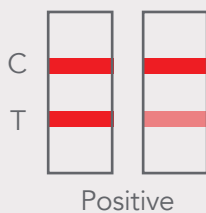

This Rapid Antigen Test Kit is an anterior nasal swab which is non-invasive and painless. It is used as a self-test for early detection and continuous self-monitoring of SARS-CoV-2/ COVID-19.

FEATURES

- Rapid results: Results are available in 15 minutes.
- Suitable for multiple scenarios: medical institution testing, screening before resuming work and school or continuous monitoring.
- High Sensitivity: Be rest assured that results are accurate due to its high sensitivity.
- Easy to Operate: Suitable for family and individual self-test of COVID-19.

DIRECTIONS: (PLEASE READ ENCLOSED INSTRUCTIONS FOR MORE DETAILS)

1. Insert the swab to the back of the nostril. Rotate 2-3 times touching the nasal wall in each nostril.
2. Tear off the sealing film and put in the specimen swab.
3. Squeeze the tube 10-15 times.
4. Leave the tube stand for 1 minute.
5. Cover the tube tip.
6. Add 3 drops of the specimen to the cassette.
7. Wait 15 minutes for the results.

STORAGE RECOMMENDATIONS:

Store at room temperature. Use immediately upon opening.