



femSense®

OVULATION TRACKER



femSense®

OVULATION TRACKER



FACT BOX

Product Code:
MD1939

Product Category:
Diagnostics

Approval:
Certified under the Medical Device Directive 93/42/EEC, Annex V

Contents:
x1 Temperature Patch

WARNING:
This patch is not a form of contraception and does not protect against sexually transmitted diseases.



Start your pregnancy journey today by scanning the QR code above or downloading the app on WWW.FLEMINGMEDICAL.IE/FEMSENSE



The Medicare femSense® Ovulation Tracker patch measures your body temperature 24/7 during your fertile window, the algorithm interprets the temperature measurements, and the app gives you daily updates on your fertility status. The app tells you which days you are most likely to get pregnant and notifies you when ovulation has occurred.

The disposable temperature patch is worn under the arm for a period of up to 7 days when a woman is most fertile, during this time it continuously measures, records and analyses body temperature data. This data is then transferred from the patch to the phone when scanned using NFC (contactless).

It is not intended for diagnostic or therapeutic use and does not display temperature curves.

FEATURES:

- No More Peeing on Sticks
- Discreet & Comfortable
- 24/7 Temperature Measurement

PRECAUTIONS:

Certain medical conditions do not allow Medicare femSense® to reliably measure ovulation. These include serious, acute or chronic diseases, nicotine or alcohol abuse, amenorrhoea, oligomenorrhoea (too infrequent menstrual bleeding), polycystic ovary syndrome (PCOS), obesity (BMI > 30), hirsutism, known allergy to components of the skin patches, medication that can influence your body temperature, hormone treatments may affect body temperature.

Being very underweight as well as very overweight can affect the reliability of Medicare femSense®. If you are not sure whether Medicare femSense® is suitable for you, please consult your doctor before beginning use.

WHAT IS IT & WHY CHOOSE IT?


The smart sensor patch is worn for a maximum of seven days – during the fertile window. This is detected based on when the last period was entered in the app. As soon as the patch is applied and activated, the 24/7 measurement begins. Precise measurement and an intelligent algorithm mean that Medicare femSense® can detect the tiny increase in body temperature caused by ovulation. Medicare femSense® can then inform the user, via the app, that ovulation has been detected and the patch can be removed. It is recommended to scan the patch at least twice a day, morning and evening.

The longer the Medicare femSense® system is used, the more accurate it becomes. By wearing the patch and closely observing several menstrual cycles, Medicare femSense® can not only detect the fertile phase, but precisely define it - individually tailored to each and every user.



DOWNLOAD THE APP

Download the Medicare femSense® app from the app store, register and get your patch ready. The algorithm can then calculate when the first patch can be applied.

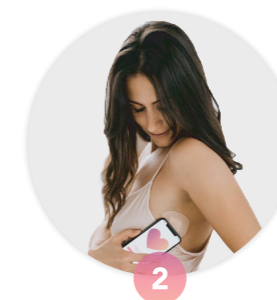

APPLY THE PATCH

The app shows when and how the patch is applied. The temperature is then measured continuously, day and night. Reading the patch with NFC (contactless) only takes seconds.



READ FERTILITY STATUS

Ovulation causes a rise in body temperature. The patch detects this rise and the app confirms that ovulation has occurred. **The patch is removed after 5-7 days.**

TRACK OVULATION

With you every step of the way, the Medicare femSense® Ovulation Tracker will help get you pregnant in the best natural, hormone free way.

Medicare femSense® is intended for any woman over the age of 18 and in childbearing age who want to track their ovulation or plan a pregnancy.

**93% success rate in detecting ovulation with over 10,000 pregnancies already recorded.*

